# SureFil<sup>®</sup> SDR<sup>®</sup> *flow+*<sup>®</sup>

## Remplissage fluidifiable

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For Dental Use Only. USA: Rx Only

### 1. PRODUCT DESCRIPTION

SureFil<sup>®</sup> SDR<sup>®</sup> *flow+* Bulk Fill Flowable is a one-component, fluoride-containing, visible light cured, radiopaque resin composite restorative material. It is designed to be used as a base in Class I and II restorations. It is also suitable as a stand-alone restorative material in non-occlusal-contact applications. SureFil<sup>®</sup> SDR<sup>®</sup> *flow+* material has handling characteristics typical of a “flowable” composite, but can be placed in 4mm increments with minimal polymerization stress. SureFil<sup>®</sup> SDR<sup>®</sup> *flow+* material has a self-leveling feature that allows intimate adaptation to the prepared cavity walls. When used as a base/liner, it is designed to be overlaid with a methacrylate based universal/posterior composite for replacing missing occlusal/facial enamel.

- 1.1 Delivery forms** (Some delivery forms may not be available in all countries) SureFil<sup>®</sup> SDR<sup>®</sup> *flow+* material is available in:
  - Predosed Compula<sup>®</sup> Tips for direct intraoral application
  - Small syringes for direct intraoral application
  - One Universal (approximates Esthet•X<sup>®</sup> HD Restorative U shade tab) and 3 VITA<sup>®</sup> (A1, A2, A3) shades

### 1.2 Composition

Barium-alumino-fluoro-borosilicate glass; Strontium alumino-fluoro-silicate glass; Modified urethane dimethacrylate resin; ethoxylated Bisphenol A dimethacrylate (EBPADMA); Triethyleneglycol dimethacrylate (TEGDMA); Camphorquinone (CQ) Photoinitiator; photoaccelerator; Butylated hydroxyl toluene (BHT); UV Stabilizer; Titanium dioxide; Iron oxide pigments fluorescing agent. Particles of inorganic filler range from 20nm to 10µm, total filler 47.3% by volume.

### 1.3 Indications

- SureFil<sup>®</sup> SDR<sup>®</sup> *flow+* material direct restorative:
  - Base in cavity Class I and II direct restorations
  - Liner under direct restorative materials – Class II box liner
  - Pit & Fissure Sealant
  - Conservative Class I restorations
  - Core Buildup
  - Class III and V restorations

### 1.4 Contraindications

- SureFil<sup>®</sup> SDR<sup>®</sup> *flow+* material is contraindicated for:
  - Use with patients who have a known hypersensitivity to methacrylate resins

### 1.5 Compatible adhesives

SureFil<sup>®</sup> SDR<sup>®</sup> *flow+* material is used following application of a suitable dentin/enamel adhesive and is chemically compatible with conventional methacrylate-based dentin/enamel adhesives including DENTSPLY adhesives designed for use with visible light cured composite restoratives (see complete Directions for Use of selected adhesive).

### 1.6 Compatible enamel replacement restoratives

SureFil<sup>®</sup> SDR<sup>®</sup> *flow+* material is used in conjunction with a suitable universal/posterior restorative material\* as an occlusal/facial enamel replacement (see Step-By-Step Instructions) and is chemically compatible with conventional methacrylate-based composite restorative materials including DENTSPLY visible light cured universal/posterior composite restorative materials designed for posterior Class I and II occlusal restorations (see complete Directions for Use of selected occlusal restorative material).

\*Enamel replacement material overlay is not required for Pit &amp; Fissure Sealant, Conservative Class I, Class III, Class V or Core Buildup applications.

## 2. GENERAL SAFETY NOTES

Be aware of the following general safety notes and the special safety notes in other sections of these directions for use.

- CAUTION**

**Safety alert symbol**

This is the safety alert symbol. It is used to alert you to potential personal injury hazards. Obey all safety messages that follow this symbol to avoid possible injury.

### 2.1 Warnings

- SureFil<sup>®</sup> SDR<sup>®</sup> *flow+* material contains polymerizable methacrylate monomers which may be irritating to skin, eyes and oral mucosa and may cause allergic contact dermatitis in susceptible persons.
- Avoid Eye Contact** to prevent irritation and possible corneal damage. In case of contact with eyes, rinse immediately with plenty of water and seek medical attention.
- Avoid Skin Contact** to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs immediately remove material with cotton and alcohol and wash well thoroughly with soap and water. In case of skin sensitization or rash, discontinue use and seek medical attention.
- Avoid Contact with Oral Soft Tissues/Mucosa** to prevent inflammation. If accidental contact occurs, immediately remove material from the tissues. Flush mucosa with plenty of water after the restoration is completed and expectorate/evacuate the water. If inflammation of mucosa persists, seek medical attention.

### 2.2 Precautions

- This product is intended to be used as specifically outlined in the Directions for Use. Any use of this product inconsistent with the Directions for Use is at the discretion and sole responsibility of the practitioner.
- Contact with saliva and blood during composite placement may cause failure of the restoration. Use of rubber dam or adequate isolation is recommended.
- Wear suitable protective eyewear, mask, clothing and gloves. Protective eyewear is recommended for patients.
- Devices marked “single use” on the labeling are intended for single use only. Discard after use. Do not reuse in other patients in order to prevent cross-contamination.
- The syringe cannot be reprocessed. To prevent the syringe and from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the syringe is handled with clean/disinfected gloves. As additional precautionary measure, syringe may be protected from gross contamination but not from all contamination by applying a protective barrier.
- SureFil<sup>®</sup> SDR<sup>®</sup> *flow+* material should extrude easily. DO NOT USE EXCESSIVE FORCE. Excessive pressure may result in unanticipated extrusion of the material or cause the Compula<sup>®</sup> Tip to eject from the Compules<sup>®</sup> Tips Gun.
- Use of Compula<sup>®</sup> Tips with the Caulk<sup>®</sup> Compules<sup>®</sup> Tips Gun is recommended.
- Tightly close syringes with original cap immediately after use. Do not allow dispensing tip to remain as a syringe cap.
- Only use the dispensing tips supplied by DENTSPLY for placement of the material.
- SureFil<sup>®</sup> SDR<sup>®</sup> *flow+*

**Compula® Tips:** The pre-dosed Compula® Tip provides the combination of a unit dose Compules® Tip with a metal cannula applicator tip.

- Load Compules® Tips Gun with pre-dosed Compula® Tip. Insert Compula® Tip into the notched opening of the gun barrel. Be certain that the collar on the Compula® Tip is inserted first.
- Remove the colored cap from the Compula® Tip. The Compula® Tip may be rotated 360° to gain the proper angle of entrance into the cavity.
- Dispense the material into the cavity preparation using a slow, steady pressure. DO NOT USE EXCESSIVE FORCE.
- To remove the used Compula® Tip, be sure that the Compules® Tips Gun plunger is pulled back completely by allowing the handle to open to its widest position. Apply a downward motion to the front end of the Compula® Tip and remove.

**Syringes:**

- Remove cap from the end of the syringe. To assure free flow of material from syringe, express a small amount onto pad, away from the patient field.
- Attach disposable, black dispensing tip to end of the syringe. Turn tip clockwise 1/4 to 1/2 turn to assure that it is fully seated. Tug on tip to be sure that it is locked into the collar of the syringe.
- Material should flow freely with gentle pressure. DO NOT USE EXCESSIVE FORCE. If more than gentle pressure is required, remove from patient field and check for obstruction.
- DISCARD AND PROPERLY DISPOSE DISPENSING TIP IMMEDIATELY AFTER USE. REPLACE ORIGINAL CAP. Do not store syringe with black dispensing tip in place. STORE ONLY WITH ORIGINAL CAP. **NOTE:** It is recommended to pull back slightly on the syringe plunger after use to prevent excessive flow of material.

#### 3.5 Placement of SureFil® SDR® flow+ material

**NOTE:** As SureFil® SDR® flow+ material is designed to conform to cavity shape, proper contact areas must be established with the matrix system selected. Material will not distend matrix band. If needed, manually distend matrix band with a suitable hand instrument such as a condenser or ball burnisher prior to and/or during light curing.

- Dispense SureFil® SDR® flow+ material directly into preparation site/tooth surface using slow, steady pressure. Begin dispensing at the deepest portion of the cavity, keeping tip close to cavity floor. Gradually withdraw tip as cavity is filled. Avoid lifting the tip out of dispensed material while dispensing to minimize air entrapment. At the completion of dispensing, wipe tip against cavity wall while withdrawing from the operative field.
- Within a few seconds, dispensed SureFil® SDR® flow+ material will self-level, eliminating the need for further manipulation with hand instruments. In case of overflow, use a flocked applicator tip wetted with residual adhesive to remove excess. Any visible air bubbles should be pierced with a clean, sharp explorer prior to curing.
- When used as a Bulk Fill Base material, most cavities may be filled in one bulk increment (up to 4mm) as needed to fill the cavity 2mm short of the occlusal cavosurface\*. In deeper preparations, place material in 4mm increments, thoroughly light curing each increment.

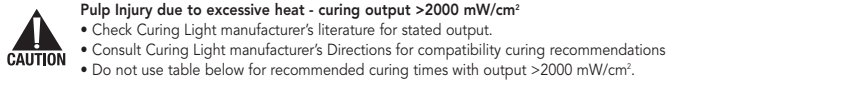
\*Alternatively, SureFil® SDR® flow+ material may be placed in a thin layer as a traditional “flowable” liner on exposed dentin.

\*When used as a Pit & Fissure Sealant, conservative Class I, Class III or V restorative or Core Buildup material, material may be placed to full contour including up to the cavosurface margin.

#### 3.6 Curing

SureFil® SDR® flow+ material is designed to be cured in increments up to a 4mm depth/thickness.

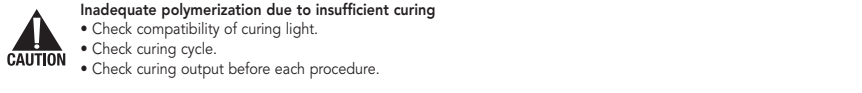
- Light cure each area of the restoration surface with a suitable visible light curing unit designed to cure materials containing camphorquinone (CQ) initiator, i.e. spectral output containing 470nm. Minimum light output must be at least 550mW/cm².

	<b>Pulp Injury due to excessive heat - curing output &gt;2000 mW/cm²</b> <ul style="list-style-type: none"><li>Check Curing Light manufacturer’s literature for stated output.</li> <li>Consult Curing Light manufacturer’s Directions for compatibility curing recommendations</li> <li>Do not use table below for recommended curing times with output &gt;2000 mW/cm².</li></ul>
--	---

#### Curing Recommendations\*

Shade	Light Output	Cure Time: 2mm	Cure Time: 4mm
Universal	Halogen and LED Lights 550-1000 mW/cm²	20*	20*
	High Power LED Lights 1000-2000 mW/cm²	10*	10*
A1, A2, A3	Halogen and LED Lights 550-1000 mW/cm²	20*	40*
	High Power LED Lights 1000-2000 mW/cm²	10*	25*

\*Refer to curing light manufacturer’s recommendations for compatibility and curing recommendations.

	<b>Inadequate polymerization due to insufficient curing</b> <ul style="list-style-type: none"><li>Check compatibility of curing light.</li> <li>Check curing cycle.</li> <li>Check curing output before each procedure.</li></ul>
--	---

#### 3.7 Completion

When used as a base/liner, proceed immediately to complete the restoration using a methacrylate based universal/posterior restorative material following manufacturer’s Directions For Use.
**Technique Tip:** In most applications, a body shade of occlusal restorative material provides the best esthetic result. In case of heavily stained dentin, use of an opaque shade may be required. Do not disturb or allow contamination of cured surface or exposed adhesive. If contamination occurs, follow adhesive manufacturer’s directions for use for re-application of adhesive. If surfaces remain undisturbed, additional application of adhesive between increments is neither necessary nor recommended.

#### 3.8 Finishing and polishing

- If contouring, finishing, and/or polishing of cured SureFil® SDR® flow+ material is necessary, use traditional rotary devices following manufacturer’s Directions For Use.
- Removal of resin flash is best accomplished with the Enhance® Finishing System of points, cups and discs. The Enhance® System will remove flash and finish restoration margins without removal of or trauma to the enamel.
- Polish final restoration using Enhance® PoGo® Polishing System or Prisma® Gloss™ and Prisma® Gloss™ Extra Fine Polishing Pastes. (See complete Directions for Use supplied with polishing product chosen.)
- All shades of SureFil® SDR® flow+ material are radio-opaque, with a 1mm radio-opacity equivalent to 2.6mm radio-opacity of aluminum. Aluminum has a radio-opacity equivalent to that of dentin. Thus 1mm of material having a radio-opacity equivalent to 1mm of aluminum has a radio-opacity equivalent to that of dentin.

## 4. HYGIENE

- Cross-contamination**
  - Do not reuse single use products. Dispose in accordance with local regulations.
  - Syringe cannot be reprocessed. Dispose of contaminated syringe in accordance with local regulations.

#### 4.1 Cleaning and disinfection

- To prevent syringes from exposure to spatter or spray of body fluids or contaminated hands, or oral tissues, use of a protective barrier is recommended to avoid syringe contamination. The use of protective barriers is an additional precautionary measure against gross contamination but not against all contamination.
- Handling with clean/disinfected gloves, syringes may be disinfected with a water-based hospital-level disinfection solution according to national/local regulations.
- Repeated disinfection may damage label. **NOTE:** Destruction of the label by vigorous wiping. Wipe syringe gently.

#### 4.2 Sterilizing the Compules® Tips Gun

*Disassembling*

- Partially close the applicator gun and place your thumb on the rear part of the hinge.
- Push upwards and lift hinge. The applicator gun is separated in two parts and the plunger is exposed.
- Remove restorative material with a soft paper tissue and alcohol of 70%.

*Sterilization*

Sterilize the applicator gun by steam autoclave (2.1-2.4 bar and 135-138°C, 30-35psi and 275-280°F). It is not recommended to submerge Compules® Tips Gun into disinfection solutions.

*Reassembling*

- After sterilization insert plunger into the applicator gun cylinder.
- Press components together and snap hinge mechanism in place.
- Do not continue to use damaged or soiled applicator guns.

## 5. LOT NUMBER AND EXPIRATION DATE

- Do not use after expiration date. ISO standard is used: “YYYY/MM”
- The following numbers should be quoted in all correspondences:
  - Reorder number
  - Lot number
  - Expiration date

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VITA is a registered trademark of Vita-Zahnfabrik H. Rauter GmbH & Co.
©2015 DENTSPLY International. All Rights Reserved. Form #51C106 (2/4/15)