




## 6 CURING

Light cure with a standard halogen or LED curing light with a minimum output of 550mW/cm<sup>2</sup>. For the universal shade, cure for 20 seconds. For shades A1, A2 and A3, cure for 40 seconds.

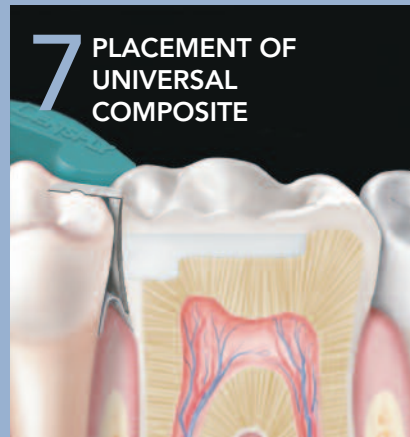


## 7 PLACEMENT OF UNIVERSAL COMPOSITE

Complete the restoration by replacing the occlusal 2mm with a methacrylate based composite (e.g. Esthet•X® HD, TPH Spectra® or Ceram•X® restoratives) following the manufacturer's instructions for use. Sculpt and contour anatomy. For most applications, a body shade provides the best esthetic results. Cure according to the manufacturer's instructions for use.



## 8 FINISHING

Contour the restoration using finishing burs or diamonds. Finish the restoration with Enhance® Finishing Cups or Points. Start finishing with a medium pressure and slowly back off to a light pressure.

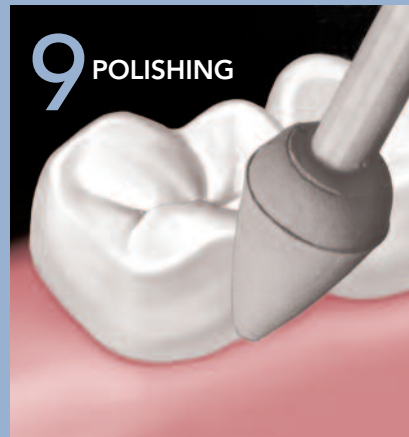

## 9 POLISHING

For a glass-like luster, polish the restoration with the PoGo® One Step Diamond Micro-Polisher System and/or Prisma® Gloss™ Polishing Paste.

DENTSPLY

## TECHNIQUE GUIDE



# 1 CAVITY PREPARATION



Prepare the cavity so no residual amalgam or restorative material is left and isolate the cavity from contamination.

# 2 PLACEMENT OF MATRIX



For Class II restorations place a matrix (e.g. AutoMatrix® or Palodent® Plus matrix systems) and wedge per the manufacturer's directions for use. Burnishing of the matrix band will improve contact and contour.

# 3 ADHESIVE APPLICATION



Refer to adhesive manufacturer's directions for pulp protection, tooth conditioning and/or adhesive application. Once the surfaces have been properly treated, they must be kept uncontaminated.

# 4 SureFil® SDR® flow + APPLICATION



Dispense SureFil® SDR® flow + material directly into preparation site using slow, steady pressure. Begin at the deepest portion of the cavity and dispense in increments up to 4mm or 2mm short of the cavosurface margin. Gradually withdraw tip as cavity is filled.

# 5 REMOVE EXCESS



At the completion of dispensing, wipe tip against cavity wall while withdrawing from the operative field. In case of overfill or excess on occlusal margins, use a flocked applicator tip wetted with residual adhesive to remove excess.

DENTSPLY

38 West Clarke Avenue  
Milford, DE 19963

1.800.LD.CAULK (532-2855)  
In Canada, call 1.800.263.1427

[www.dentsply.com](http://www.dentsply.com)

Rx Only

For Dental Use Only



Caution, Consult

Instructions for Use

©2014 DENTSPLY International. All Rights Reserved.  
Form #51C108 (R 12/23/14)